

SANAAY

NAADI PARIKSHA SIMPLIFIED


Naadi Pariksha is an ancient method to measure ayurvedic tridosha, understand patient's physical and psychological state and provide remedies. Using a pulse oximeter, SaNaaY makes the entire process simple and repeatable to the physicians.

Using SaNaaY physician's can:

- (1) Register themselves
- (2) Register Patients
- (3) Assess patient's prakriti
- (4) Assess Vikriti and imbalance using Sahaj Naadi Yantra
- (5) Store and retrieve results from cloud
- (6) Store manual naadi result
- (7) Generate, print and share reports



How to download app

- Go to www.sanaay.com or scan the QR code
- Click on download for doctors to download the app
- Install that into your android device to get started

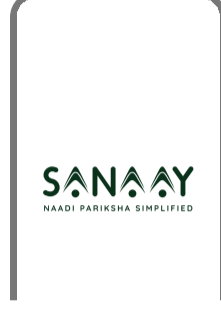


Register doctor and clinic

- Enter doctor detail
- Enter your clinic details
- Verify using OTP

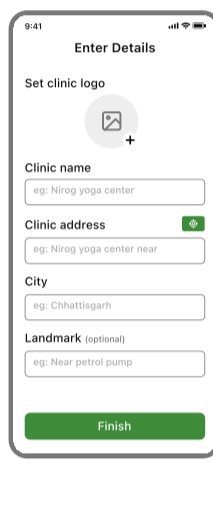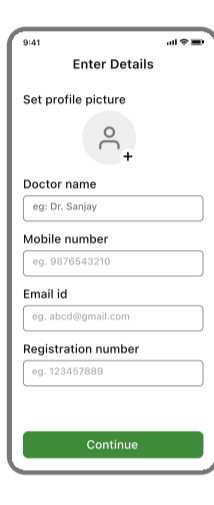




How to register SaNaaY via Bluetooth

- Add battery to the naadi yantra and turn it on
- Turn on bluetooth on your android smartphone
- Pair your nadi yantra to your android smartphone

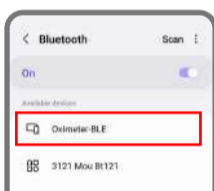


Add patients

- Add patients: Take prakriti quiz or doctor enters prakriti manually

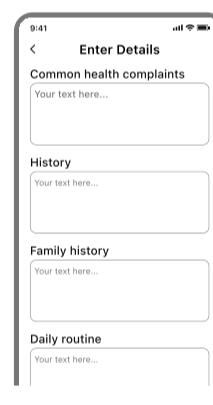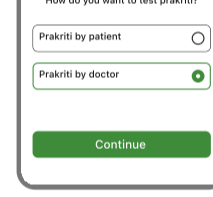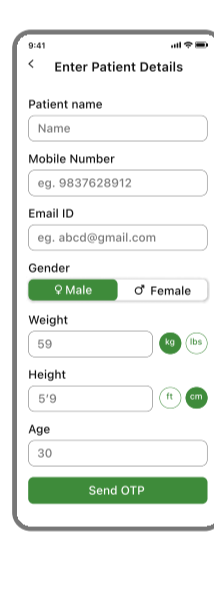



How to measure

- Choose naadi by SaNaaY while testing vikriti of the patient
- Press continue and follow the instructions
- Review test result and Select Recommendations

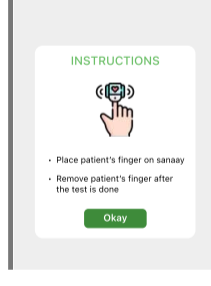


Other features

- Book appointments in advance
- View patient health history
- Two different engines to diagnose vikriti and prakriti
- Select recommendations for patients based on their doshas
- Generate report, share and print report

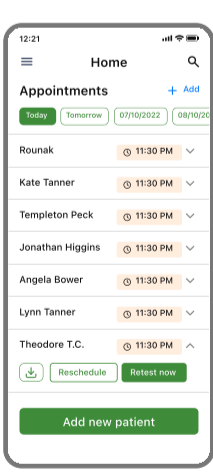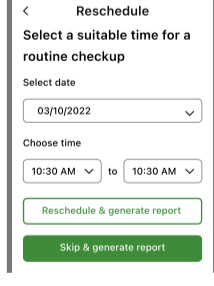

Web Link to download detailed product manual

You can download the detailed product manual by scanning the given QR code or you can download it from www.sanaay.com

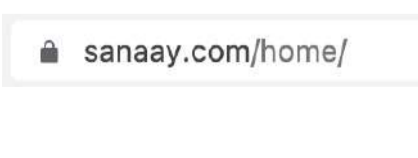

OR



SAHAJ NAADI YANTRA

|| सहज नाडी यंत्र ||

Please refer to this manual before using the product